

# Haematology Counter

## Operating Instructions

### Lights

50   
 100   
 200

50   
 100   
 200


50   
 100   
 200

50   
 100   
 200

50   
 100  **Flashing**  
 200

50   
 100  **Flashing**  
 200

50   
 100   
 200

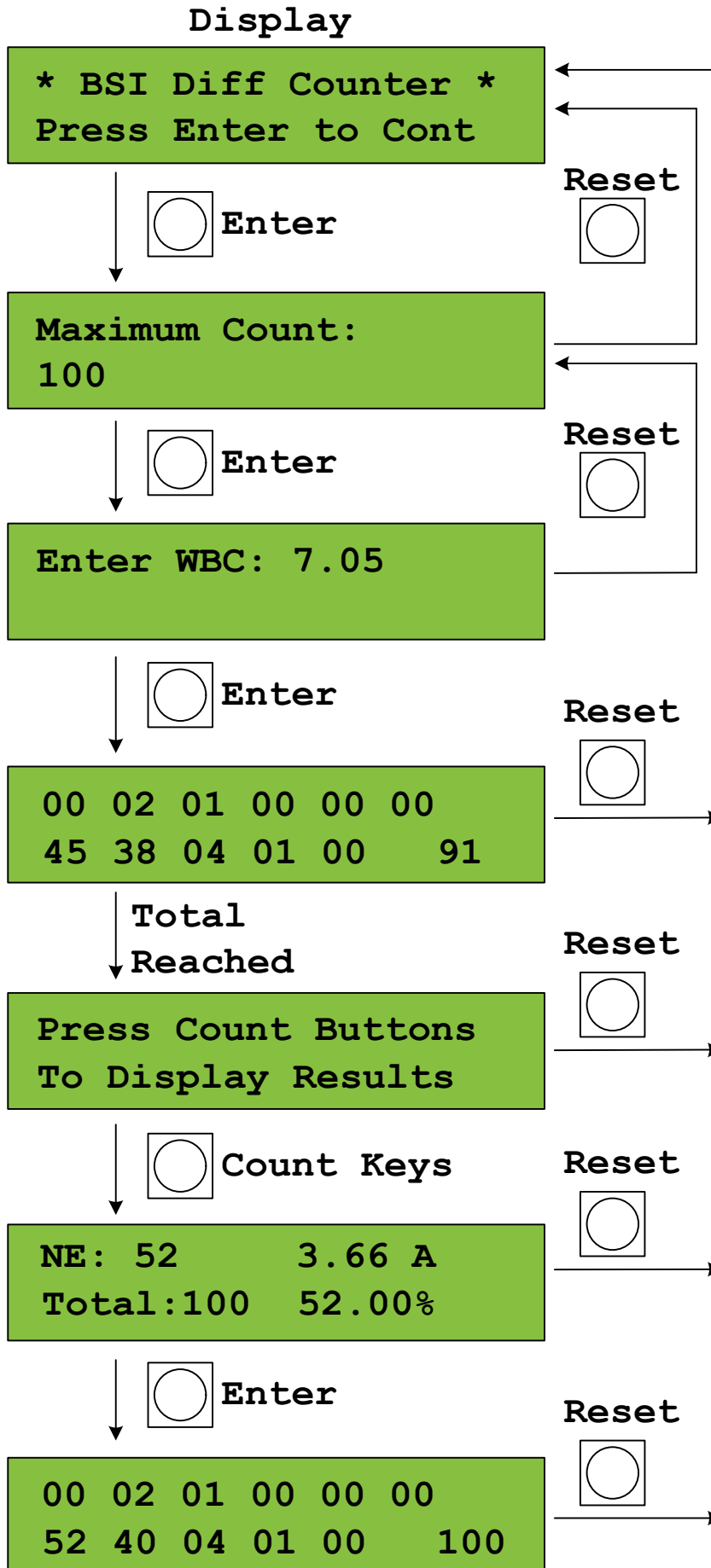

Press **Enter** to start the counting process.

Default maximum count is always 100 indicated by the lights on far left of the keys. The maximum count can be changed to 200 or 50 or back to 100 by pressing any key except **Reset** or **Enter**.

Enter the White Blood Cell Count. This is done by using the count keys which double as numeric values, while the **NRBC** key doubles as the Point. Counting can commence without entering a WBC value by pressing **Enter** immediately.

Start counting by pressing the required count keys for each cell type observed. An overview of each count is displayed with a total count on the bottom right of the display. The **NRBC** key does not increment the overall total. The **ATYPS** key increments the overall total but the number of **ATYPS** is not displayed in the overview.

Once the pre-selected total count has been reached a buzzer will sound and the corresponding count light will flash. If the count keys are now pressed the totals for that indicator will be displayed. Repeatedly pressing the same count key toggles between the totals display and overview display.

The 'A' symbol represents the absolute value and is calculated relative to the WBC value that was entered at the start of the counting process. This will be 0 if no WBC value was entered.

To continue counting to a higher value press the **Enter** key at this point and continue with the count.